Instructions for use:

Single-use filtering face-piece respirators

**Important:** Single-use respirators are classified according to one of three categories: FFP1 NR, FFP2 NR and FFP3 NR. Before reading the following information, please check the category to which the respirator belongs. This is indicated on the packaging and on the respirator. If the mask is marked D, it has passed the dolomite clogging test and is suitable for use in a very dusty environment.

It is user's responsibility to ensure that the respirator provides the necessary level of protection for the type and concentration of the contaminant(s) in the area where the respirator is intended for use.

Directions for use:

- Fit the respirator and check the leak-tightness BEFORE entering the contaminated area.
  - Wear the respirator for the whole duration of exposure to contaminants.
  - Use the respirator in accordance with the applicable health and safety regulations.
  - Discard the respirator and replace it with a new one if
    - The respirator is removed while in a contaminate area
    - Excessive clogging on the respirator causes breathing difficulty or discomfort
    - The respirator becomes damaged
    - (For respirators protecting against vapours) The smell of vapours becomes detectable
  - Leave the contaminated area if dizziness, irritation or other distress occurs.
  - For single use only. No maintenance necessary. Do not store and/or reuse after the single use.
  - Discard the respirator after a single use (one shift maximum)
  - Keep unused respirators in their closed box and store in a dry non-contaminated area between -20°C and +40°C at relative humidity under 75%.
- Notified Body: BSI Group, The Netherlands B.V. (Notified Body number 2797), Say Building John M. Keynesplein 9, 1066 EP Amsterdam

Fitting instructions:

1. Hold the respirator in one hand with the nosepiece at your fingertips and pull the headband with the other hand.
2. Press the respirator against your face with the nosepiece on the bridge of your nose.
3. Place the top band high on the back of your head. Move the bottom band over your head and position it below your ears.
4. Pull the head straps with both hands at the same time to obtain a comfortable fit.
5. Using both hands, mould the nosepiece to the shape of your nose.
6. Check for leak-tightness (facial fit) as follows:
  - a) - place both hands over the respirator and exhale sharply
    - positive pressure should be noticeable inside the respirator
    - if you feel air escaping around the edges, please readjust the respirator by tightening the nosepiece and/or increasing tension of the headbands
    - repeat the procedure until satisfactory face fit has been achieved.
  - b) Test the fit of a respirator with an exhalation valve. Cup both hands over the respirator and inhale vigorously. If no negative pressure is to be felt inside the respirator, please readjust the position of the respirator according to steps 2 to 6 until receiving a better fit.

Limitations of use:

- Do not use these respirators, enter or stay in an area where:
- oxygen concentration is less than 17% (UK – 19%)
  - Contaminants or their concentrations are unknown or immediately dangerous to life or health
  - Particulate concentrations exceed levels fixed by applicable health and safety regulations or protection factor x NFP – whichever is lower (Nominal Protection Factors: 4.5 for FFP1 NR, 12.5 for FFP2 NR, 50 for FFP3 NR)
  - Gases and/or vapours are present, except for respirators designed to protect against vapours, in which case the vapour concentrations should not exceed NPF. Respirators protecting against organic vapours should not be used as protection against acid vapours and vice versa.

Warning:

Do not use for fire-fighting. These respirators do not supply oxygen. Do not use in oxygen-deficient atmospheres e.g. tanks or other poorly ventilated areas (please see "Limitations of use") Do not use in explosive atmospheres. For use only by trained and qualified personnel. Requirements for leak-tightness are unlikely to be achieved if the respirator is worn over a beard or facial stubble. When transporting this product, please retain in the original packaging and keep away from mechanical and chemical hazards.

Storage and transportation

Store in accordance with manufacturer's instructions, see packaging.

temperature range 

maximum relative humidity 

please read the user instructions carefully 

Please kindly find the validity date on the bottom of the packaging box

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MANUFACTURER IDENTIFICATION:

Type: FFP1 NR D : FFP2 NR D : FFP3 NR D

NR is to be worn for a single shift only  
D: Meet the Dolomite clogging test  
CE Standard considered: EN 149:2001+A1:2009  
CE Marking : CE 2797

Technical information

| Tests                                        | Type | FFP1 NR   | FFP2 NR   | FFP3 NR |
|----------------------------------------------|------|-----------|-----------|---------|
| Total Inward Leakage                         |      | <22%      | <8%       | <2%     |
| Penetration                                  |      | <20%      | <6%       | <1%     |
| Carbon dioxide content of inhalation air (%) |      | <1%       | <1%       | <1%     |
| Inhalation resistance 30 l/min (mbar)        |      | <0.6 mbar | <0.7 mbar | <1 mbar |
| Inhalation resistance 95 l/min (mbar)        |      | <2.1 mbar | <2.4 mbar | <3 mbar |
| Exhalation resistance 160 l/min (mbar)       |      | <3 mbar   | <3 mbar   | <3 mbar |

PPE's controlled by:

BSI Group The Netherlands B.V, Say Building John M. Keynesplein 9, 1066 EP Amsterdam

Our filtering half masks are in conformity with the PPE Regulation (EU) 2016/425.

EU Declaration of Conformity is available to download from our website under the Europe of CERTIFICATE.

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# VERTICAL FOLD FLAT DISPOSABLE MASK

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